

SIEMENS MAGNETOM Product syngo MR B17

\\USER\knee\_20min\_0002\20min\_protocols\_NYU\_27.05.09\ankle\_8ch\_ok\pd\_tse\_fs\_p2

TA: 2:56 PAT: 2 Voxel size: 0.4x0.3x3.0 mm Rel. SNR: 1.00 SIEMENS: tse

Properties

Prio Recon	Off
Before measurement	
After measurement	
Load to viewer	On
Inline movie	Off
Auto store images	On
Load to stamp segments	On
Load images to graphic segments	Off
Auto open inline display	On
Start measurement without further preparation	On
Wait for user to start	Off
Start measurements	single

Routine

Slice group 1	
Slices	35
Dist. factor	10 %
Position	Isocenter
Orientation	Coronal
Phase enc. dir.	R >> L
Rotation	0.00 deg
Phase oversampling	50 %
FoV read	140 mm
FoV phase	100.0 %
Slice thickness	3.0 mm
TR	4860 ms
TE	52 ms
Averages	1
Concatenations	1
Filter	Distortion Corr.(2D), Prescan Normalize, Image Filter
Coil elements	FA

Contrast

MTC	Off
Magn. preparation	None
Flip angle	150 deg
Fat suppr.	Fat sat.
Fat sat. mode	Weak
Water suppr.	None
Restore magn.	On
-----	
Averaging mode	Long term
Reconstruction	Magnitude
Measurements	1
Multiple series	Each measurement

Resolution

Base resolution	512
Phase resolution	75 %
Phase partial Fourier	Allowed
Trajectory	Cartesian
Interpolation	Off
-----	
PAT mode	GRAPPA
Accel. factor PE	2
Ref. lines PE	35
Matrix Coil Mode	Auto (Triple)
Reference scan mode	Integrated
-----	
Image Filter	On
Intensity	Medium

Edge Enhancement	3
Smoothing	3
Unfiltered images	Off
Distortion Corr.	On
Mode	2D
Unfiltered images	Off
Unfiltered images	Off
Prescan Normalize	On
Normalize	Off
B1 filter	Off
Raw filter	Off
Elliptical filter	Off

Geometry

Multi-slice mode	Interleaved
Series	Interleaved
-----	
Special sat.	None
-----	
Set-n-Go Protocol	Off
Table position	H
Table position	0 mm
Inline Composing	Off
-----	
Tim CT mode	Off

System

Body	Off
FA	On
-----	
Positioning mode	FIX
MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	H >> F
Coil Combine Mode	Adaptive Combine
AutoAlign	---
Auto Coil Select	Default
-----	
Shim mode	Standard
Adjust with body coil	Off
Confirm freq. adjustment	Off
Assume Silicone	Off
? Ref. amplitude 1H	0.000 V
Adjustment Tolerance	Auto
Adjust volume	
Position	Isocenter
Orientation	Coronal
Rotation	0.00 deg
F >> H	140 mm
R >> L	140 mm
A >> P	116 mm

Physio

1st Signal/Mode	None
-----	
Dark blood	Off
-----	
Resp. control	Off

Inline

Subtract	Off
Std-Dev-Sag	Off
Std-Dev-Cor	Off

# SIEMENS MAGNETOM Product syngo MR B17

Std-Dev-Tra	Off
Std-Dev-Time	Off
MIP-Sag	Off
MIP-Cor	Off
MIP-Tra	Off
MIP-Time	Off
Save original images	On

---

## Sequence

Introduction	On
Dimension	2D
Compensate T2 decay	Off
Reduce Motion Sens.	Off
Contrasts	1
Bandwidth	257 Hz/Px
Flow comp.	No
Allowed delay	0 s
Echo spacing	13 ms

---

Define	Turbo factor
Turbo factor	9
Echo trains per slice	34
RF pulse type	Low SAR
Gradient mode	Fast