

# SIEMENS MAGNETOM TrioTim syngo MR B17

\\USER\20min\_protocols\elbow\elbow\fl2d\_cor

TA: 3:02    PAT: Off    Voxel size: 0.4x0.3x2.0 mm    Rel. SNR: 1.00    SIEMENS: gre

## Properties

Prio Recon	Off
Before measurement	
After measurement	
Load to viewer	On
Inline movie	Off
Auto store images	On
Load to stamp segments	Off
Load images to graphic segments	Off
Auto open inline display	Off
Start measurement without further preparation	On
Wait for user to start	Off
Start measurements	single

## Routine

Slice group 1	
Slices	30
Dist. factor	20 %
Position	Isocenter
Orientation	Coronal
Phase enc. dir.	F >> H
Rotation	90.00 deg
Phase oversampling	25 %
FoV read	120 mm
FoV phase	100.0 %
Slice thickness	2.0 mm
TR	500 ms
TE	10.00 ms
Averages	1
Concatenations	1
Filter	Prescan Normalize, Elliptical filter
Coil elements	FS

## Contrast

MTC	Off
Magn. preparation	None
Flip angle	25 deg
Fat suppr.	None
Water suppr.	None
SWI	Off
-----	
Averaging mode	Short term
Reconstruction	Magnitude
Measurements	1
Multiple series	Each measurement

## Resolution

Base resolution	384
Phase resolution	75 %
Phase partial Fourier	Off
Interpolation	Off
-----	
PAT mode	None
Matrix Coil Mode	Auto (CP)
-----	
Image Filter	Off
Distortion Corr.	Off
Unfiltered images	On
Prescan Normalize	On
Normalize	Off
B1 filter	Off
Raw filter	Off

Elliptical filter    On  
Mode    Inplane

## Geometry

Multi-slice mode	Interleaved
Series	Interleaved
-----	
Saturation mode	Standard
Special sat.	None
-----	
Set-n-Go Protocol	Off
Table position	H
Table position	22 mm
Inline Composing	Off
-----	
Tim CT mode	Off

## System

Body	Off
FS	On
-----	
Positioning mode	REF
MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	F >> H
Coil Combine Mode	Adaptive Combine
AutoAlign	---
Auto Coil Select	Default
-----	
Shim mode	Tune up
Adjust with body coil	Off
Confirm freq. adjustment	Off
Assume Silicone	Off
? Ref. amplitude 1H	0.000 V
Adjustment Tolerance	Auto
Adjust volume	
Position	Isocenter
Orientation	Transversal
Rotation	0.00 deg
R >> L	350 mm
A >> P	263 mm
F >> H	350 mm

## Physio

1st Signal/Mode	None
Segments	1
-----	
Tagging	None
Dark blood	Off
-----	
Resp. control	Off

## Inline

Subtract	Off
Liver registration	Off
Std-Dev-Sag	Off
Std-Dev-Cor	Off
Std-Dev-Tra	Off
Std-Dev-Time	Off
MIP-Sag	Off
MIP-Cor	Off
MIP-Tra	Off
MIP-Time	Off

# SIEMENS MAGNETOM TrioTim syngo MR B17

Save original images	On
-----	
Wash - In	Off
Wash - Out	Off
TTP	Off
PEI	Off
MIP - time	Off
-----	
-----	
MapIt	None
Contrasts	1

## Sequence

Introduction	On
Dimension	2D
Phase stabilisation	Off
Asymmetric echo	Off
Bandwidth	260 Hz/Px
Flow comp.	No
Allowed delay	0 s
-----	
RF pulse type	Normal
Gradient mode	Fast
Excitation	Slice-sel.
RF spoiling	On