SIEMENS





FNH/Adenoma

Distinction with TWIST-VIBE¹

www.siemens.com/body-mri-cases

Mike Notohamiprodjo, M.D.

Associate Chair

Department of Diagnostic and Interventional Radiology

University Hospital Tübingen

Germany

Philipp Kazmierczak, M.D.

Department of Clinical Radiology

University Hospitals Munich

Germany

Background

Patient data




Female

54

/ear

Patient history

A number of intrahepatic masses were found on sonograph during a routine checkup. There were no symptoms.

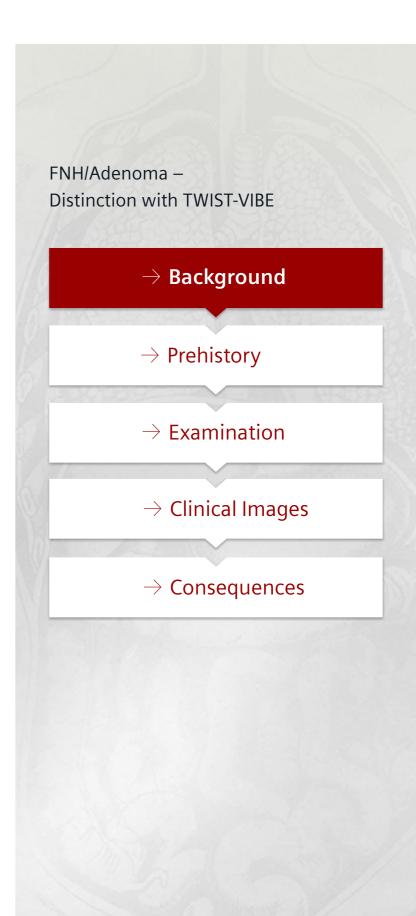


Prehistory

Prior diagnostic examinations

A number of intrahepatic masses showed up on sonograph during a routine checkup. Out-patient MR examination classified the masses as possible adenomas/FNHs. No arterial phase was performed in this scan. It was therefore not possible to classify the lesion. A definitive differential diagnosis was not possible due to missing diagnostic confidence with the results.

Differential diagnosis


Without having additional contrasts in the arterial phase, the differential diagnosis would be adenoma vs. cyst and FNH vs. vascular pseudo lesions.

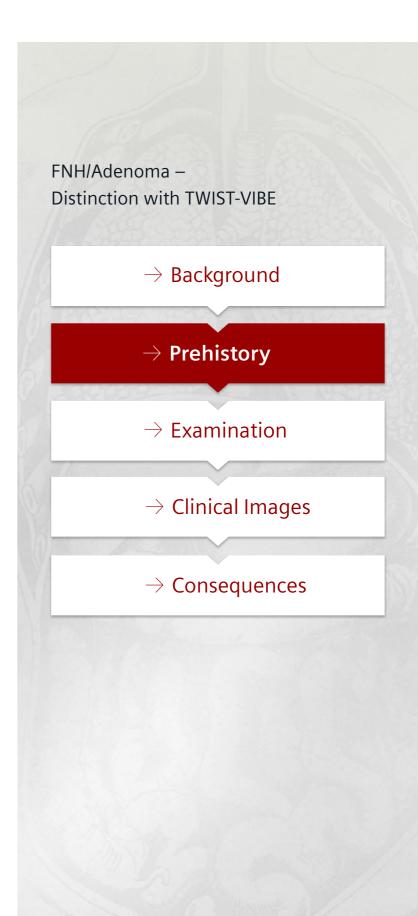


Examination

MR imaging findings


The MR examination clearly shows one fatty lesion in segment 4b on the pure Dixon-VIBE fat image (image 1), which can therefore be classified as adenoma. It is rather hypo-vascularized in the contrast-enhanced TWIST-VIBE (image 2). The other visible lesions show significant contrast uptake in the arterial phases of TWIST-VIBE (image 3). Adding all available background information, the lesions are most probably FNHs. The Eovist late phase already indicates the increased uptake of the FNHs and the weaker but present retention of the adenomas (image 4). Potentially malignant lesions were not identified, with a high degree of certainty.

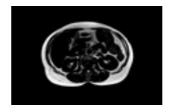


ightarrow Fat image

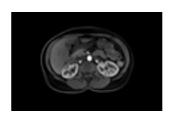


 \rightarrow TWIST-VIBE (14s)

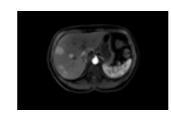



 \rightarrow TWIST-VIBE (14s)

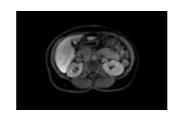



→ TWIST-VIBE (Eovist late phase)

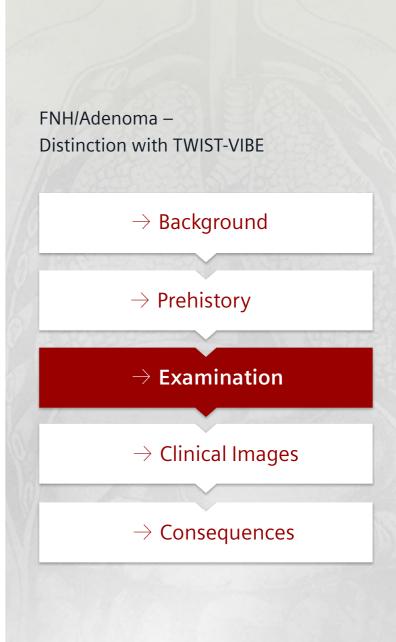


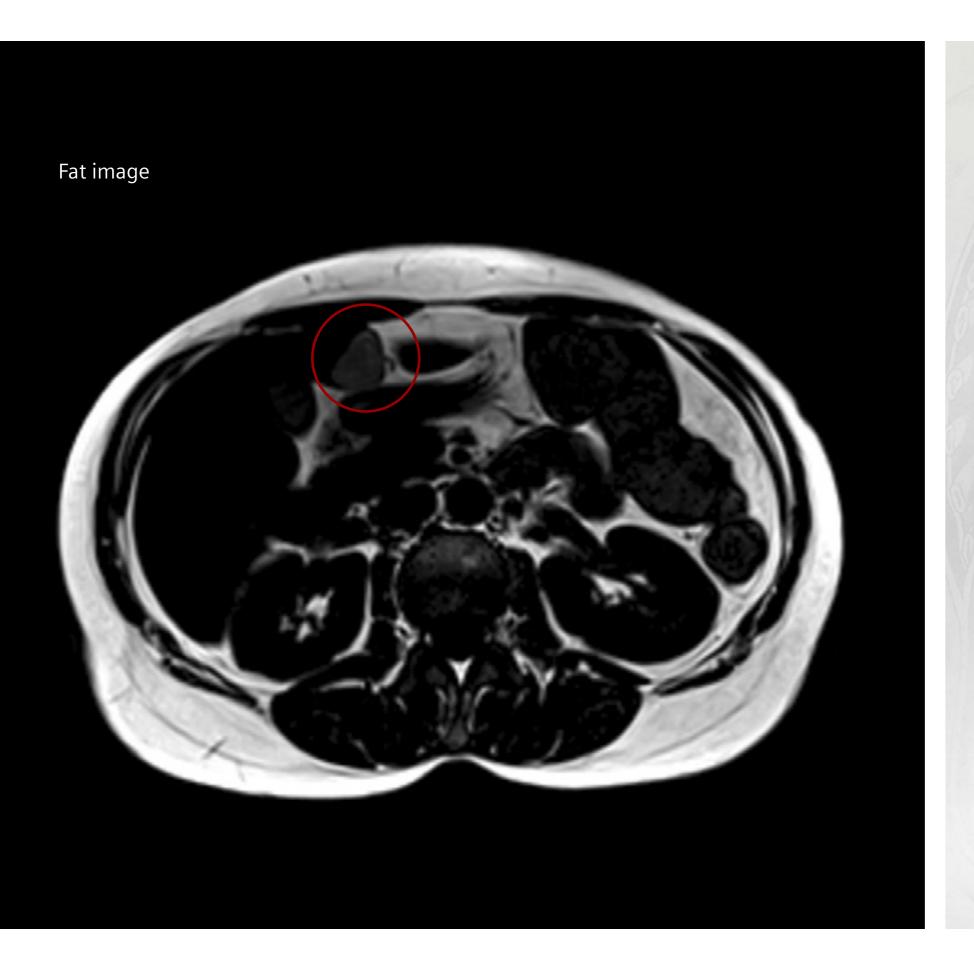


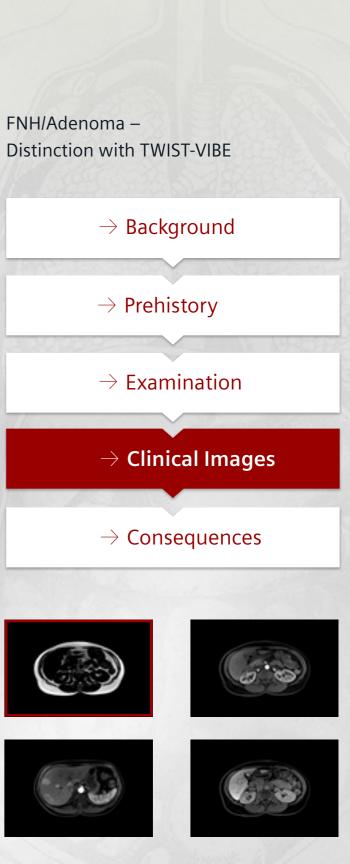


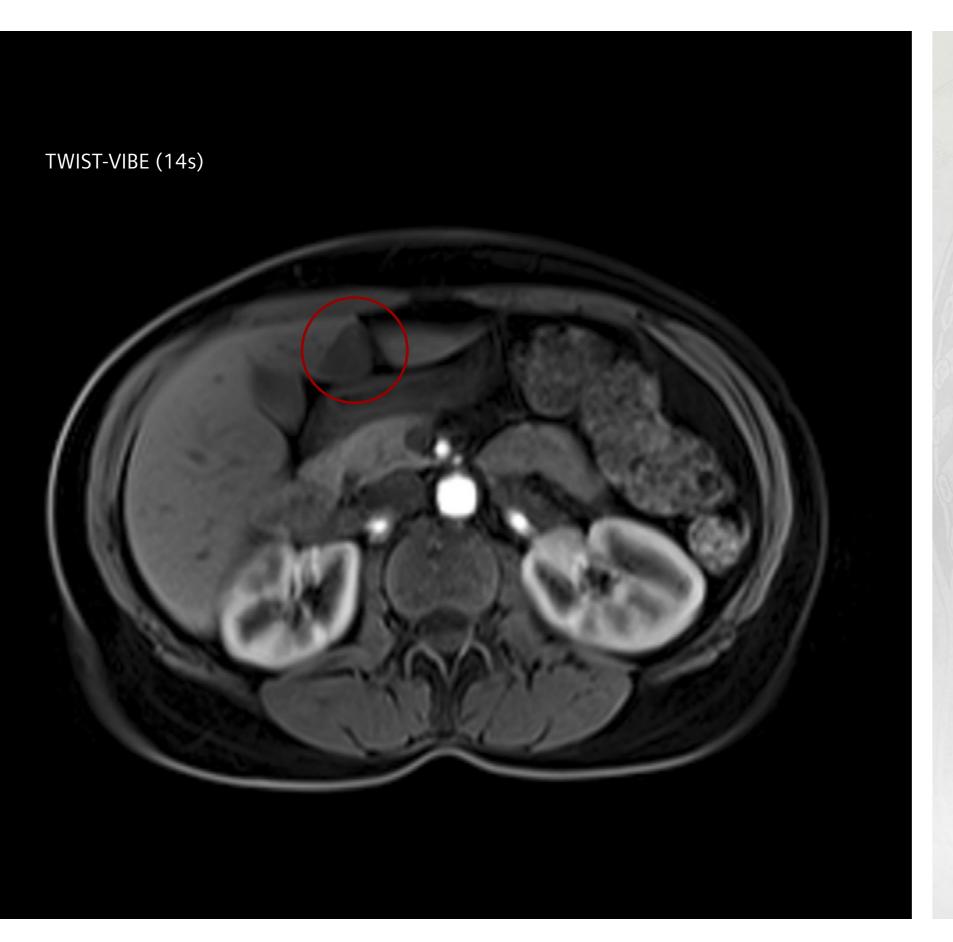


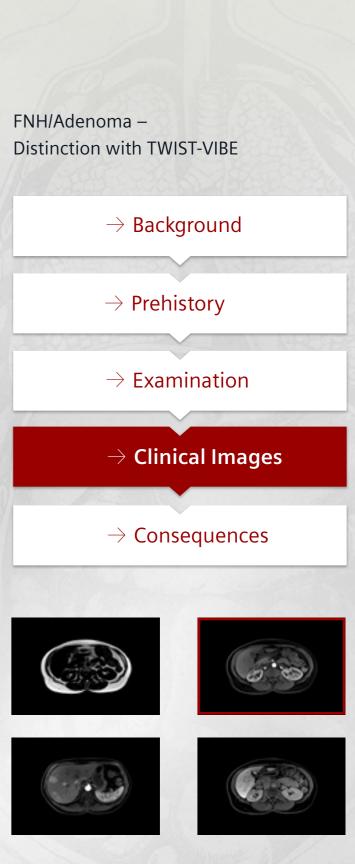

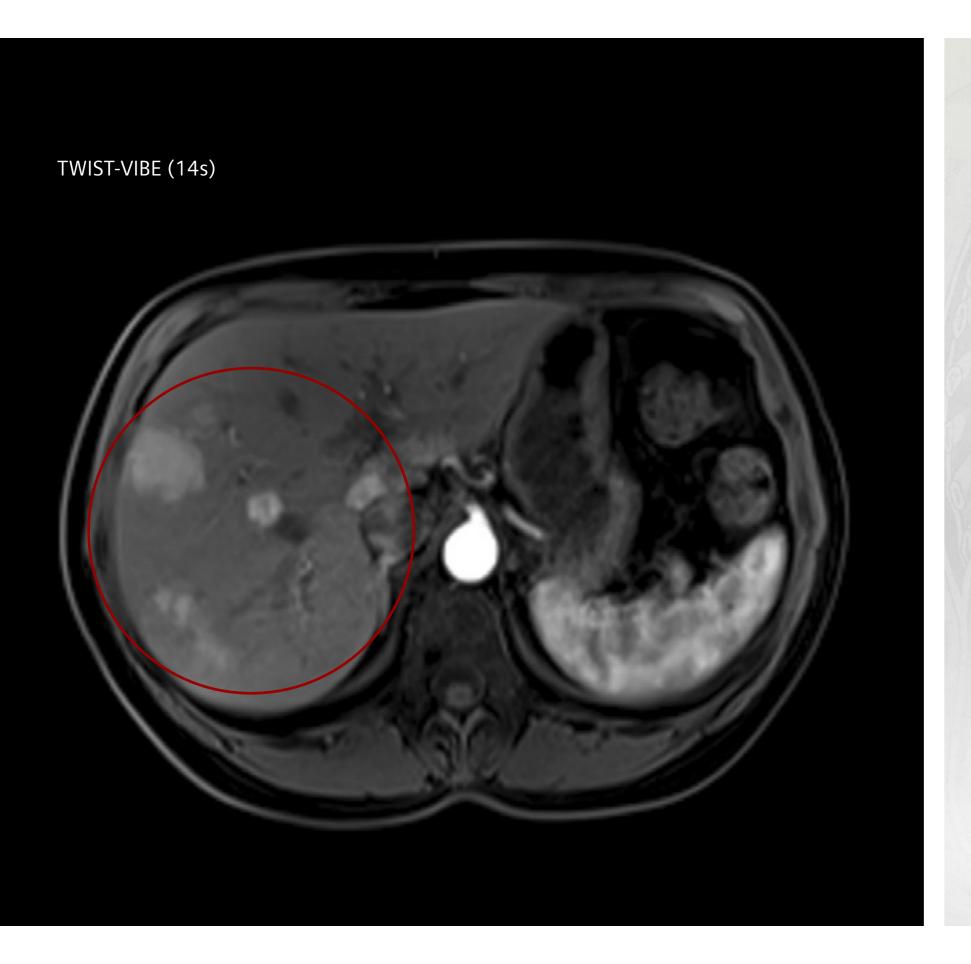

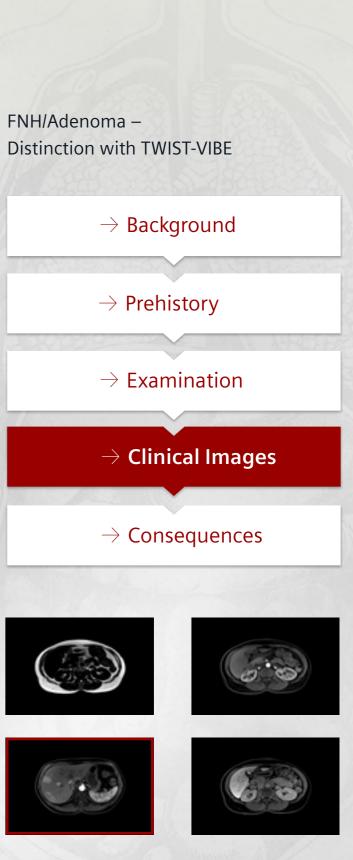

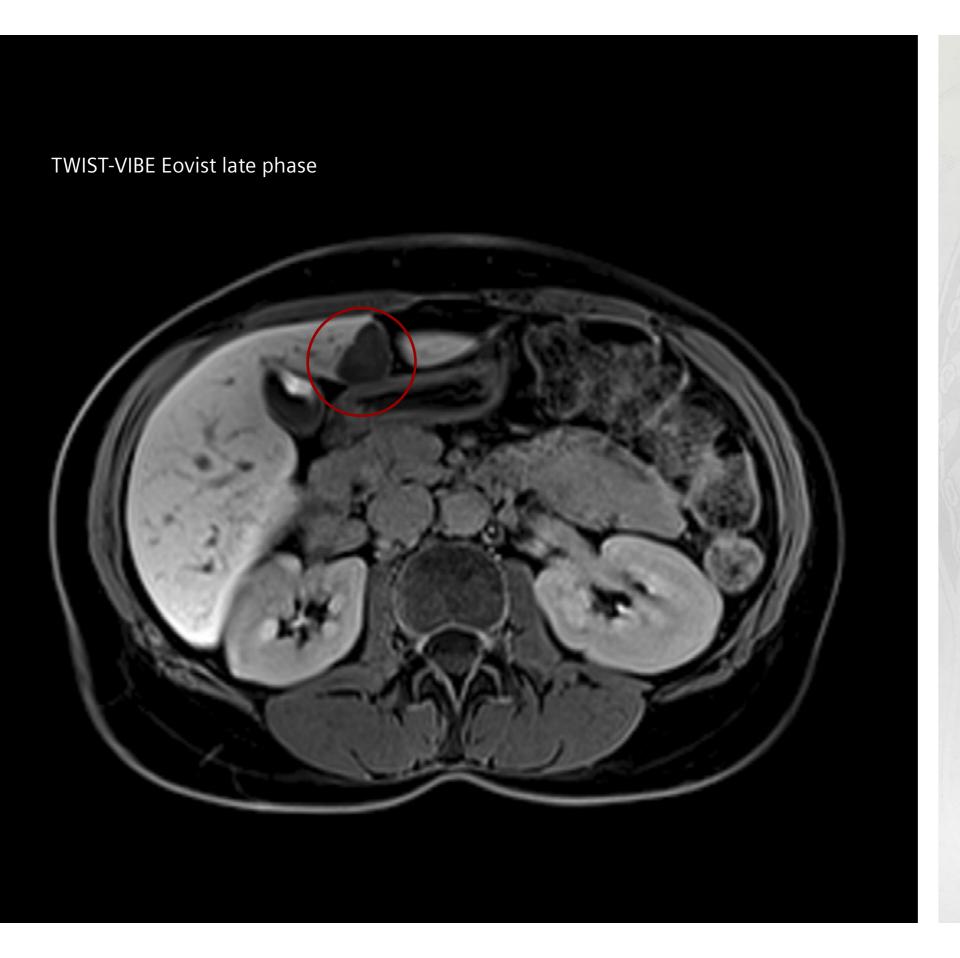

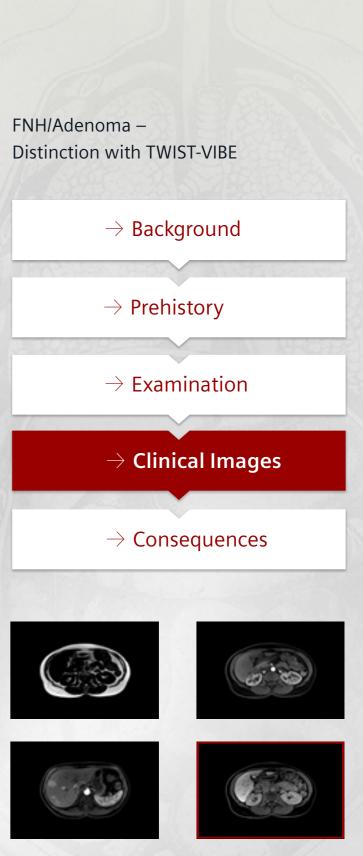


Consequences

Consequences and procedure

At somewhat increased AFP, the adenomas are further monitored with MRI, as there is a low risk of degeneration, which can be effectively examined using MRI including TWIST-VIBE.

Therapeutic and prognostic relevance

MRI provides a higher added value in many clinical scenarios compared to CT and ultrasound, especially in the context of appropriate treatment selection and outcome monitoring. In this case, it was possible to answer all clinical questions with a comprehensive MRI examination of the liver. TWIST-VIBE gives more information compared to the older VIBE sequence - namely, multiple arterial phases to make sure to identify the best phase for the characterization of the lesion. In addition, the maximum peak of the lesion enhancement can be detected. The patient will be monitored in the future by using MRI – no radiation dose, clear results.

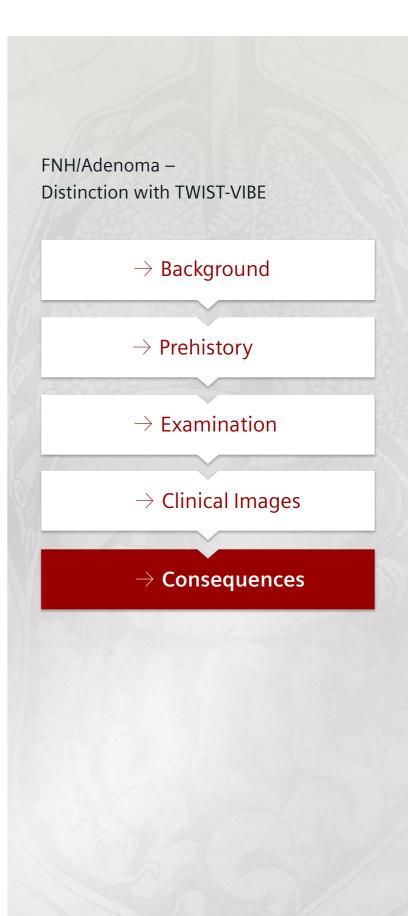

On account of certain regional limitations of sales rights and service availability, we cannot guarantee that all products included in this brochure are available through the Siemens sales organization worldwide. Availability and packaging may vary by country and is subject to change without prior notice. Some/All of the features and products described herein may not be available in the United States.

The information in this document contains general technical descriptions of specifications and options as well as standard and optional features which do not always have to be present in individual cases, and which may not be commercially available in all countries. Due to regulatory reasons their future availability cannot be guaranteed. Please contact your local Siemens organization for further details.

Siemens reserves the right to modify the design, packaging, specifications, and options described herein without prior notice. Please contact your local Siemens sales representative for the most current information.

Note: Any technical data contained in this document may vary within defined tolerances. Original images always lose a certain amount of detail when reproduced.

Local Contact Information

Asia/Pacific:

Siemens Medical Solutions Asia Pacific Headquarters The Siemens Center 60 MacPherson Road Singapore 348615 Phone: +65 6490 6000

Canada:

Siemens Canada Limited
Healthcare Sector
1550 Appleby Lane
Burlington, ON L7L 6X7, Canada
Phone +1 905 315-6868

Europe/Africa/Middle East:

Siemens AG, Healthcare Sector Henkestr. 127 91052 Erlangen, Germany Phone: +49 9131 84-0

Latin America:

Siemens S.A., Medical Solutions Avenida de Pte. Julio A. Roca No 516, Piso C1067 ABN Buenos Aires, Argentina Phone: +54 11 4340-8400

USA:

Siemens Medical Solutions USA, Inc. 51 Valley Stream Parkway Malvern, PA 19355-1406, USA Phone: +1 888 826-9702 ¹ FREEZEit and TWIST-VIBE are not commercially available in all countries. Due to regulatory reasons their future availability cannot be guaranteed. Please contact your local Siemens organization for further details.

© 2014 Siemens AG

www.siemens.com/body-mri-cases

Global Business Unit

Siemens AG
Medical Solutions
Magnetic Resonance
Henkestraße 127
DE-91052 Erlangen
Germany

Phone: +49 9131 84-0

www.siemens.com/healthcare

Global Siemens Headquarters

Siemens AG Wittelsbacherplatz 2 80333 Muenchen Germany Global Siemens Healthcare Headquarters

Siemens AG Healthcare Sector Henkestraße 127 91052 Erlangen Germany

Phone: +49 9131 84-0 www.siemens.com/healthcare