



Beginning an Exam

1. Press **NEW PATIENT** in the upper left corner of keyboard.
2. Enter patient information using **TAB** to get to next datafield.
3. Select transducer and exam type from drop down menus.
4. Select **OK** or **ESCAPE/PRIORITY** to begin scanning.

Optimizing Image

<i>To adjust for...</i>	<i>Do this...</i>
Gain	Press TGO key
Less noise in image	Press THI key
Brightness level	Rotate 2D knob
More focal zones	Press FOCUS knob & turn to adjust position
Increase penetration	Toggle far left key down (MULTIHERTZ on soft key menu)
Exam type	Press EXAM key, SET key to select
Transducer type	Press TRANSDUCER button to change automatically

Measure

1. **FREEZE**, choose measurement type from soft key menu, select **CALIPER**, place caliper using trackball, press **SET**. Roll next caliper with trackball, then press **SET**.
2. Press **CALIPER** again to get another set.
3. To remove **CALIPERS**, unfreeze the image.

ACUSON X150™ ultrasound system

Quick Start Card

SIEMENS

Measure for OB Report Measurements

Same as Step 1 under “**Measure**”, then select **MEASUREMENT** from left menu, double click **SET** to display on screen and transfer to report.

Measure Heart Rate using M-mode

Press **M** key and place cursor. Press **M** key again, freeze, set calipers and use trackball to select **HEART RATE**.

Texting on Image

<i>To...</i>	<i>Do this...</i>
Type text	Press TEXT button on control panel
Set text home	Press HOME SET (soft key menu)
Erase Text	Press TEXT key then the CLEAR SCREEN soft key
Activate Arrow	Press ARROW key on top of keyboard

Transfer Studies to CD

Insert blank CD, press **PATIENT BROWSER**, select patient name, select **EXPORT TO CD**, press **EJECT**.

Transfer Studies to USB Device

Insert USB device, press **PATIENT BROWSER**, select **STUDY SCREEN** and then **HD**. Select **USB** in **EXPORT/IMPORT** section, go to **STUDIES**, and select **EXPORT**. Press **ESCAPE** to leave **PATIENT BROWSER**.

Legal Manufacturer

Siemens Medical Solutions USA, Inc.
Ultrasound
1230 Shorebird Way
Mountain View, CA 94043 USA
Telephone: +1-888-826-9702

www.siemens.com/ultrasound