

Example 1: NGSP Unit Reporting only  
System Settings:

**Settings** 10:45 PM 04/30/2005

HbA1c Reporting Units

Primary                      Secondary

NGSP                       None     NGSP

IFCC                         IFCC

mono-S                       mono-S

JCCLS                         JCCLS

Result display on screen:

**HbA1c Test** 10:45 PM 04/30/2005

HbA1c Patient Test Result - NGSP

**HbA1c = 13.5% +**

HbA1c Reporting Units(1)	NGSP
Sequence No.	XXX
Sample ID	ABC123
Patient ID	MX762092183
Last Name	HOLLENZOLLBEREN