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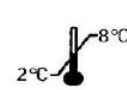
<b>SIEMENS</b>		SMN 10445968
<b>Protein S Ac</b>		
<b>CONTENTS</b>		<b>REF</b> OPAP03
6 x → 1 mL	<b>DEFICIENT</b>	<b>04</b> siemens.com/eifu
2 x → 2 mL	<b>REAGENT</b>	
2 x → 5 mL	<b>ACTIVATOR</b>	
		<b>LOT</b> 507000
		<b>LOT</b> 547800
		<b>LOT</b> 547900
<b>For USA</b> <span style="float: right;"><b>RxOnly</b></span> <b>Intended Use:</b> Determination of the functional activity of protein S in human plasma. <b>Ingredients:</b> Human plasma, activated human protein C, Russell's Viper venom (0.75 mg/L), plant phospholipids (50 mg/L) <b>Precaution:</b> Contains human source material		 <b>IVD</b> <b>CE</b>

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Document Categories. The Document Library provides customers with a single point-of-access to technical documents for our entire portfolio of medical devices, assays, test kits and Healthcare IT offerings.

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## N/T Protein Control LC

N/T PROT CONTROL LC

Revision bar indicates update to previous version.

**BN II / BN ProSpec® System**

**Intended Use**

N/T Protein Control LC is an assayed intralaboratory quality control for assessment of precision and analytical bias in immunonephelometric determination of IgG in urine and CSF, IgA in CSF, IgM in CSF, transferrin in urine, albumin in urine and CSF, α<sub>2</sub>-macroglobulin in urine, Ig/L-chains (κ-type and λ-type) in urine, α<sub>1</sub>-microglobulin in urine and β-trace protein using the BN II and BN ProSpec® System.

**Reagent**

**Materials provided**

N/T PROT CONTROL LC REF OQLW  
 3 x → 1.0 mL N/T PROT CONTROL LC, N/T Protein Control LC

**Composition and Standardization**

N/T Protein Control LC is a lyophilized control prepared of polygeline and rabbit albumin with urinary and serum proteins of human origin.  
 The concentration of the listed analytes is calibrated against different standard preparations and is lot-specific.  
 The assigned values and confidence intervals are given in the enclosed table. The values can be read into the BN ProSpec® System via Lot Data CD (REF OVLP).  
 Preservative: sodium azide (< 1 g/L)

**Warnings and Precautions**

For *in-vitro* diagnostic use.  
 Contains sodium azide (< 1 g/L) as a preservative. Sodium azide can react with copper or lead pipes in drain lines to form explosive compounds. Dispose of properly in accordance with local regulations.